
HOGGI[®]



ZIP

Spare parts catalogue

en

Index

1	Modellidentification and basic equipment	
1.1	Modellidentification	3
1.2	Basic equipment	3
2	Assemblies	
2.1	Assembly overview	4
3	Spare parts	
3.1	Front strut	5
3.2	Rear strut	6
3.3	Part list rear strut	7
3.4	Push bar	8
3.5	Support lever	9
3.6	Crossbrace	10
3.7	Spread strud	11
3.8	Back rest frame	12
3.9	Part list back rest frame	13
3.10	Seat frame	14
3.11	Nylon cover	15
3.12	Foot rest over view	16
3.13	Foot board reciever	17
3.14	Foot board assembly	18
3.15	Footboard system continuos	19
3.16	Back rest angle extention up to 5°	20
3.17	Castor fork	21
3.18	Wheel options	22
3.19	Swivel lock for castor fork	23
3.20	Attendent break two hand use	24
3.21	Grab rail	25
3.22	Therapy tray height adjustable	26
3.23	Therapy tray	27
3.24	Tie down kit size 1	28
3.25	Tie down kit size 2	29
3.26	Canopy incl. adapter and rain cover	30
3.27	Part list Canopy incl. adapter and rain cover	31
3.28	Abduktion block	32

1 Model identification

1.1 Model identification

The label with the serial number, is placed on the back frame.

Please give us the Serial number with every spare part order, to ensure a correct delivery.

The diagram shows a label with three identical sections. Each section contains the following information:

- HOGGI** logo
- HOGGI GmbH**
Eulerstraße 27
D-56235 Ransbach-Baumbach
- MADE IN GERMANY**
- ZIP** (Reha-Faltbuggy)
- Zuladung: 50 kg**
- CE** mark

To the right of each section is a vertical barcode with the text **SN 2018XXXXX**. Lines point from the labels to the corresponding parts of the label:


- Colour code
- Item number
- Size
- Serial number
- CE-Identification
- identification identification
- Type designation / Productname
- Maximum user weight / Maximum load
- Manufacturer
- Manufactur
- Year of manufactur

1.2 Basic configuration

The basic configuration includes:


- Reinforced nylon seat and back cover
- Footrest - infinitely height-adjustable, angle-adjustable and fold-away
- Ball bearing wheels front and rear
- Park break

2 Component overview

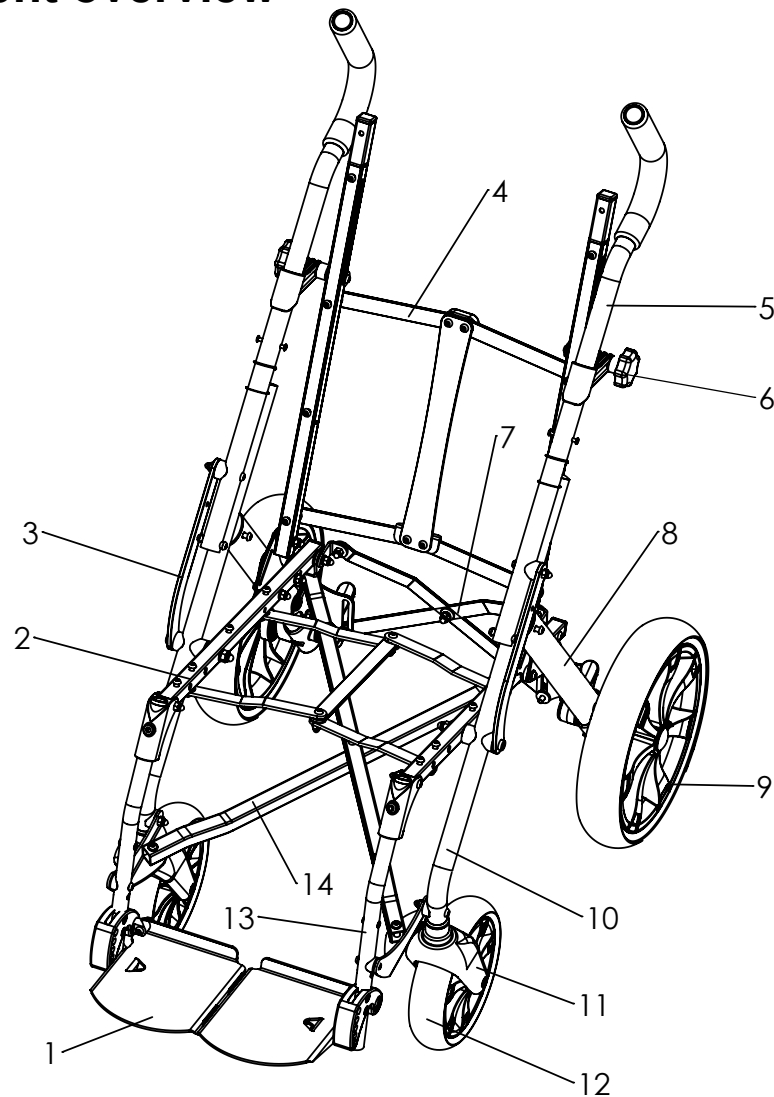


Pos	Artikelnummer	Artikelbezeichnung	Seite
1	321X-2310-015	Foot board	18
2	321X-1700-052	Seat frame	14
3	3210-1400-052	Support Lever	9
4	321X-1800-052	Back frame	12
5	3210-1310-052	Push handle	8
6	3210-8500	Recline angle extension 5°	20
7	321X-1240-052	Locking scissors	11
8	3210-1210-052	Rear strut	6
9	1221-xxxx	Rear wheels	22
10	321X-1110-052	Front strut	5
11	1281-0007E	Castor fork	21
12	1221-2048	Front wheels	22
13	321X-2320	Foot board receiver	17
14	321X-1600-052	Cross strut	10

Spare parts

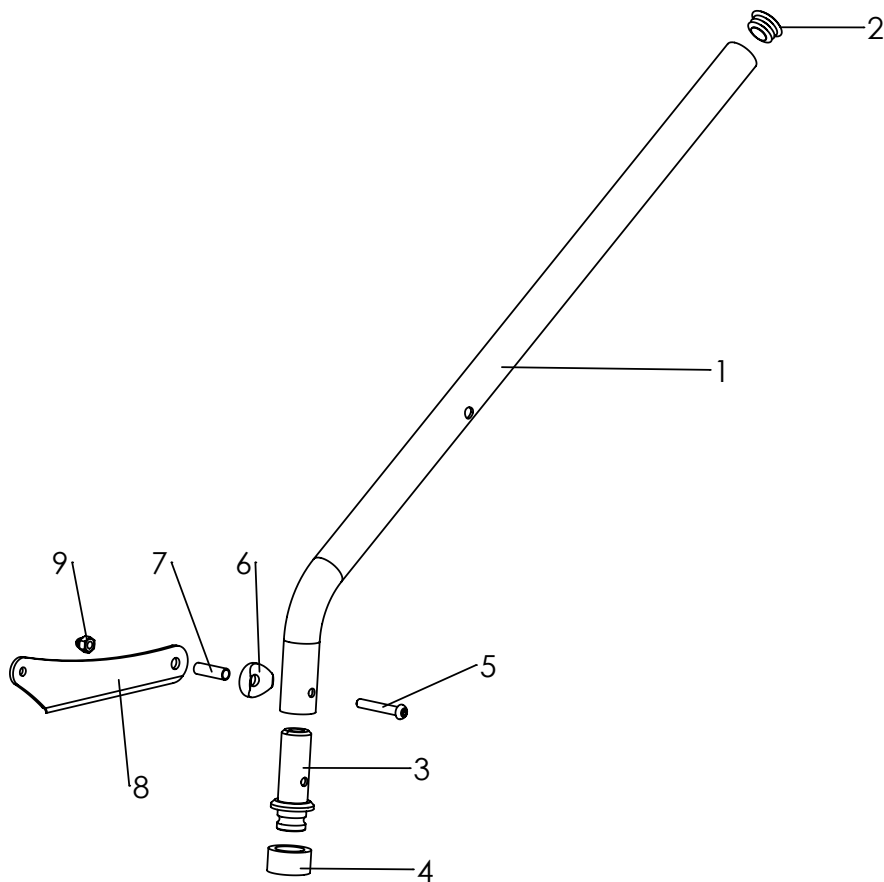

Pos	Article number	Articel description	Quantity	Unit
1-9	3210-1110E-052	Front strut size 1 left complete, black	1	Pc.
	3210-1110E-052	Front strut size 1 right complete, black	1	Pc.
	3213-1110E-052	Front strut size 2 left complete, black	1	Pc.
	3213-1110E-052	Front strut size 2 right complete, black	1	Pc.
1	3210-1111-001	Front strut left size 1, black	1	Pc.
	3210-1112-001	Front strut right size 1, black	1	Pc.
	3213-1111-001	front strut left, size 2, black	1	Pc.
	3213-1112-001	front strut right, size 2, black	1	Pc.
2	1141-0009	Ribbed insert SFL 25x1-2,5	1	Pc.
3	3210-1114	Front fork retainer size 1	1	Pc.
	3213-1114	Front fork retainer size 2	1	Pc.
4	3202-1151	Cover ring	1	Pc.
5	1029-0640	Button head socket screw M6x40	1	Pc.
6	1111-0012	Saddle washer PA black 24x8.4x3	1	Pc.
7	1114-0025	Bushing zp 8x0,8x27	1	Pc.
8	3210-1116-015	Joint angle left size 1	1	Pc.
	3210-1117-015	Joint angle right size 1	1	Pc.
	3213-1116-015	Joint angle left size 2	1	Pc.
	3213-1117-015	Joint angle right size 2	1	Pc.
9	1044-0006	Hexagonal domed cap nut self-locking M6	1	Pc.


Spare parts

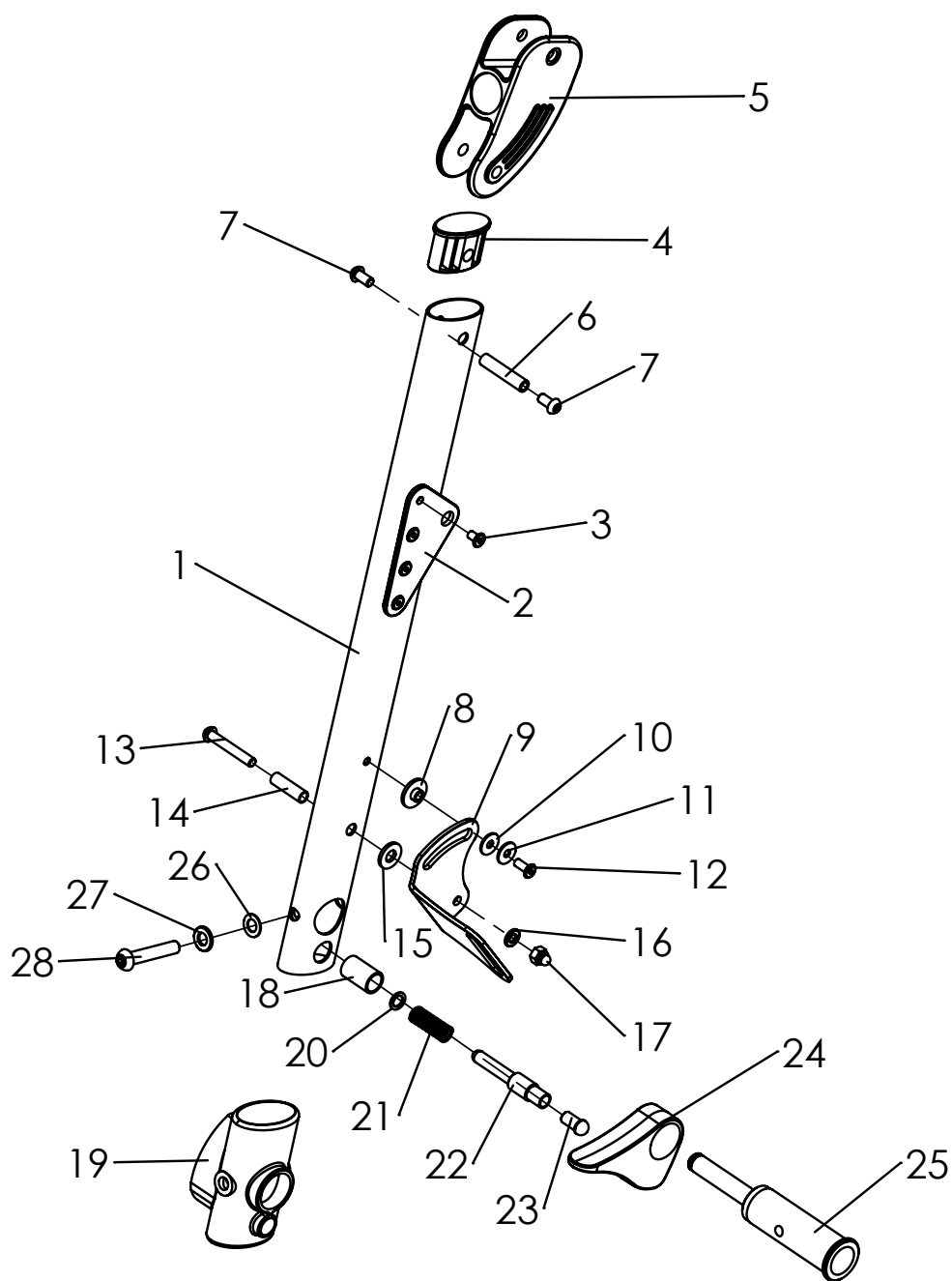

Spare parts


Pos	Article number	Articel description	Quantity	Unit
1-28	3210-1210E-052	Rear strut, left complete, black	1	Pc.
	3210-1220E-052	Rear strut, left complete, black	1	Pc.
1	3210-1211-052	Rear strut	1	Pc.
2	3210-1212-015	Junction plate	1	Pc.
3	1062-0508	Blind rivet F5x8	4	Pc.
4	3201-1270	Plug	1	Pc.
5	3201-1500	Slider	1	Pc.
6	1114-0028	Threaded bushing 8xM6x38	1	Pc.
7	1029-0612	Button head socket screw M6x12	2	Pc.
8	3210-1219	Guide bushing	1	Pc.
9	3210-1217-015	Strut guide left	1	Pc.
	3210-1218-015	Strut guide right	1	Pc.
10	1113-0059	Washer PA black 15x5,3x1,5	1	Pc.
11	1055-0005	Washer A5,3	1	Pc.
12	1062-0516	Blind rivet F5x16	1	Pc.
13	1029-0640	button head socket screw M6x40	1	Pc.
14	1114-0025	Bushing zp 8x0,8x27	1	Pc.
15	1113-0035	Washer PA 18x8,4x2 black	1	Pc.
16	1055-0006	Washer A6,4	1	Pc.
17	1044-0006	Hexagonal domed cap nut self-locking 986 M6	1	Pc.
18	3210-1216	Guid bushing	1	Pc.
19	3210-1215	Connector	1	Pc.
20	1059-0008	Sealing Ring DIN 7603 8x12x1,0 Copper	1	Pc.
21	1131-0034	Compression spring D-180	1	Pc.
22	3210-1930	Brake bolt	1	Pc.
23	3201-1340	Slide insert	1	Pc.
24	3210-1910	Brake lever, left side	1	Pc.
	3210-1920	Brake lever, right side	1	Pc.
25	3210-1214	Axle	1	Pc.
26	1051-0008	Washer A8,4	1	Pc.
27	1052-0008	Spring washer B8	1	Pc.
28	1029-0845	Button head socket screw M8x45	1	Pc.

Spare parts

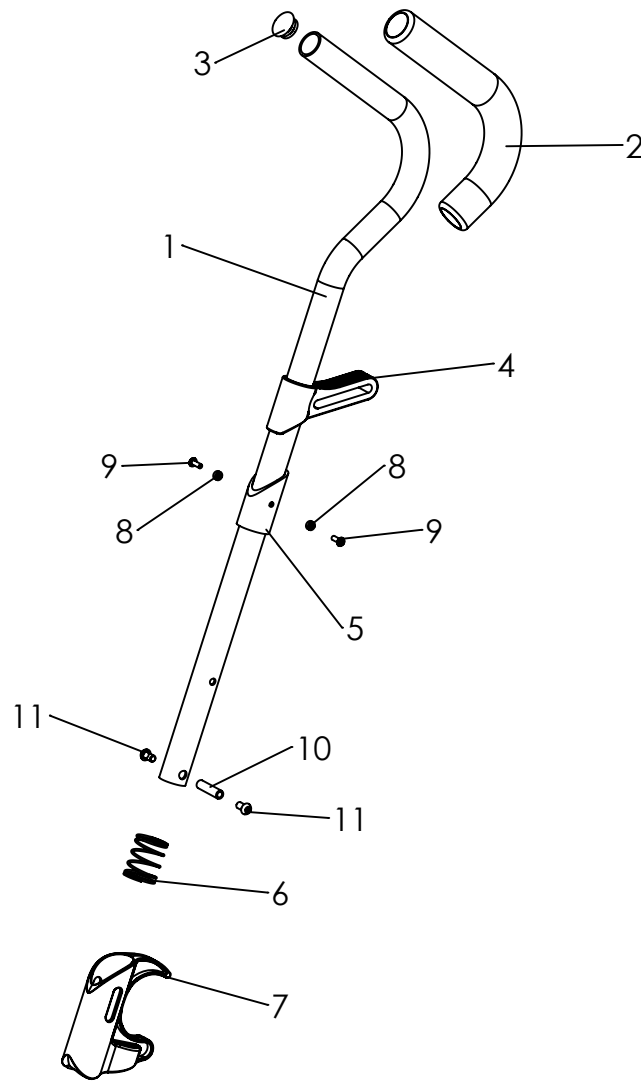


Pos	Article number	Articel description	Quantity	Unit
1-10	3210-1310E-052	Push bar complete, black	2	Pc.
1	3210-1311-052	Handlebar, black	2	Pc.
2	1143-0001	Handle grip EVA black d30 x 260	2	Pc.
3	1141-0009	Ribbed insert SFL 25x1-2,5	2	Pc.
4	3210-1840	Back glider	2	Pc.
5	3201-1440	Sliding sleeve	2	Pc.
6	1131-0033	Compression spring Vd-232	2	Pc.
7	3201-1430	Folding safety device	2	Pc.
8	1112-0010	Spacer PA black 8x4,3x3	4	Pc.
9	1062-0412	Blind rivet 4x12	4	Pc.
10	1114-0027	Threaded bushing 8xM6x32	2	Pc.
11	1029-0612	Button head socket screw M6x12	2	Pc.

Spare parts

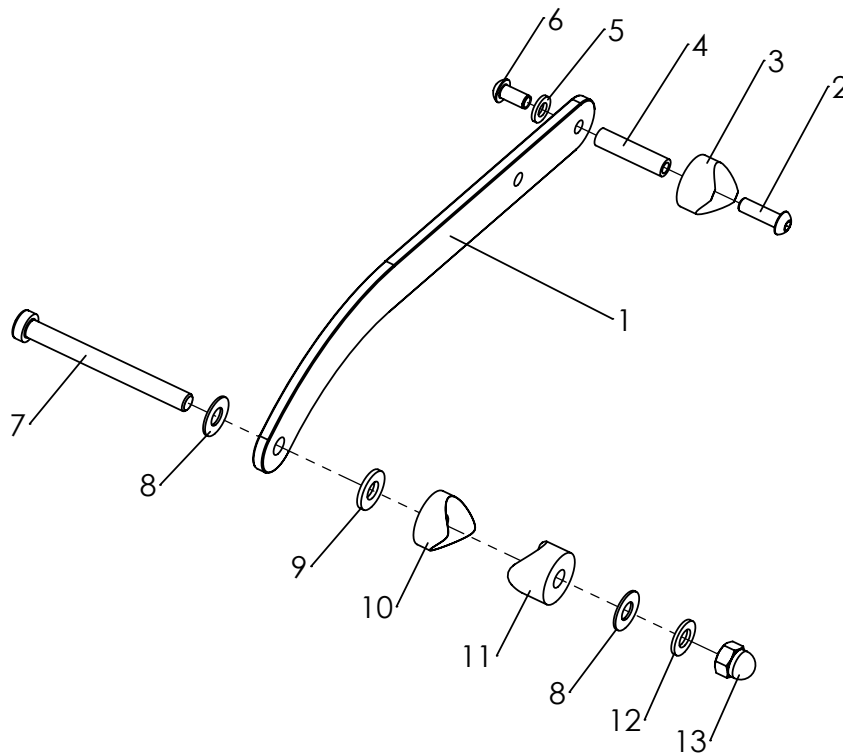


Pos	Article number	Articel description	Quantity	Unit
1-13	3210-1400E-052	Support lever size 1 black, complete	1	Pc.
	3213-1400E-052	Support lever size 2 black, complete	1	Pc.
1	3210-1410	Support lever size 1	1	Pc.
	3213-1410	Support lever size 2	1	Pc.
2	1029-0620	Button head socket screw M6x20	1	Pc.
3	1111-0011	Saddle washer PA 24x8.4x8	1	Pc.
4	1114-0065	Threaded bushing 8xM6x33,2	1	Pc.
5	1051-0006	Washer A6,4	1	Pc.
6	1029-0612	Button head socket screw M6x12	1	Pc.
7	1022-0880	Hexagon socket head cap screw M8x80	1	Pc.
8	1113-0003	Washer PA 18x8,4x1 black	2	Pc.
9	1113-0035	Washer PA 18x8,4x2 black	1	Pc.
10	1111-0014	Saddle washer PA black 24x8.4x6	1	Pc.
11	1111-0013	Saddle washer PA black 24x8.4x10	1	Pc.
12	1051-0008	Washer A8,4 zp DIN/ISO 125 A8,4	1	Pc.
13	1044-0008	Hexagonal domed cap nut self-locking M8	1	Pc.

Spare parts

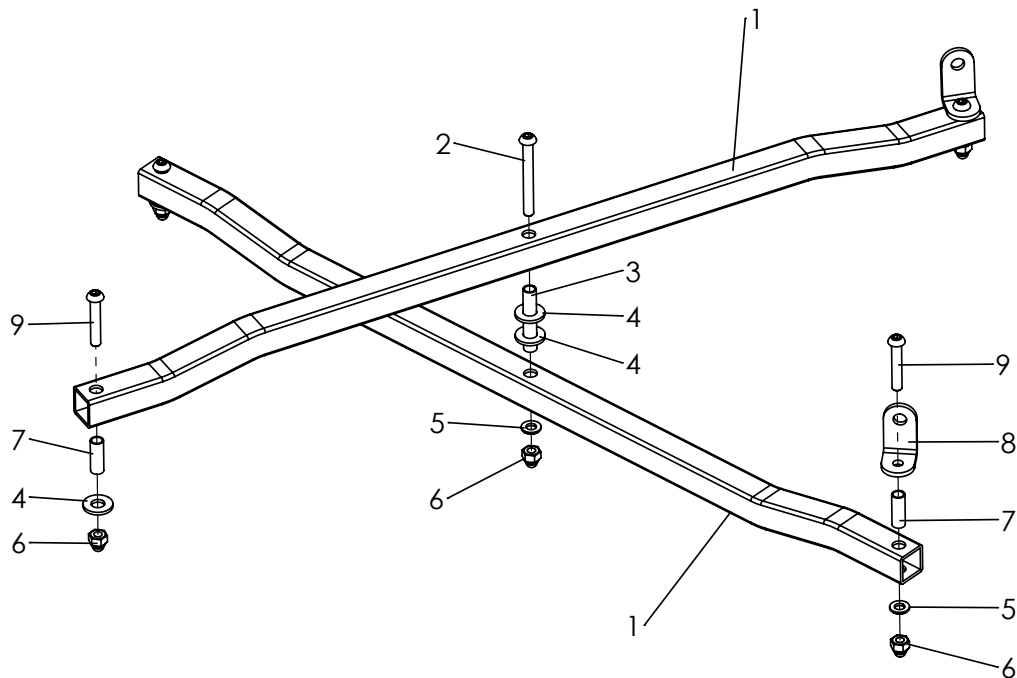


Pos	Article number	Articel description	Quantity	Unit
1-9	3210-1600E-052	Crossbrace size 1, complete, black	1	Pc.
	3213-1600E-052	Crossbrace size 2, complete, black	1	Pc.
1	3210-1610-052	Scissor arm, size 1, black	2	Pc.
	3213-1610-052	Scissor arm, size 2, black	2	Pc.
2	1029-0655	Button head socket screw M6x55	1	Pc.
3	1114-0053	Bushing zp 8x0,8x44,5	1	Pc.
4	1113-0035	Washer PA 18x8,4x2 black	4	Pc.
5	1051-0006	Washer A6,4 zp DIN/ISO 125 A6,4	3	Pc.
6	1044-0006	Hexagonal domed cap nut self-locking M6	5	Pc.
7	1114-0012	Bushing zp 8x0,8x22,5	4	Pc.
8	3210-1715	Joint angle	2	Pc.
9	1029-0635	Button head socket screw M6x35	4	Pc.

Spare parts

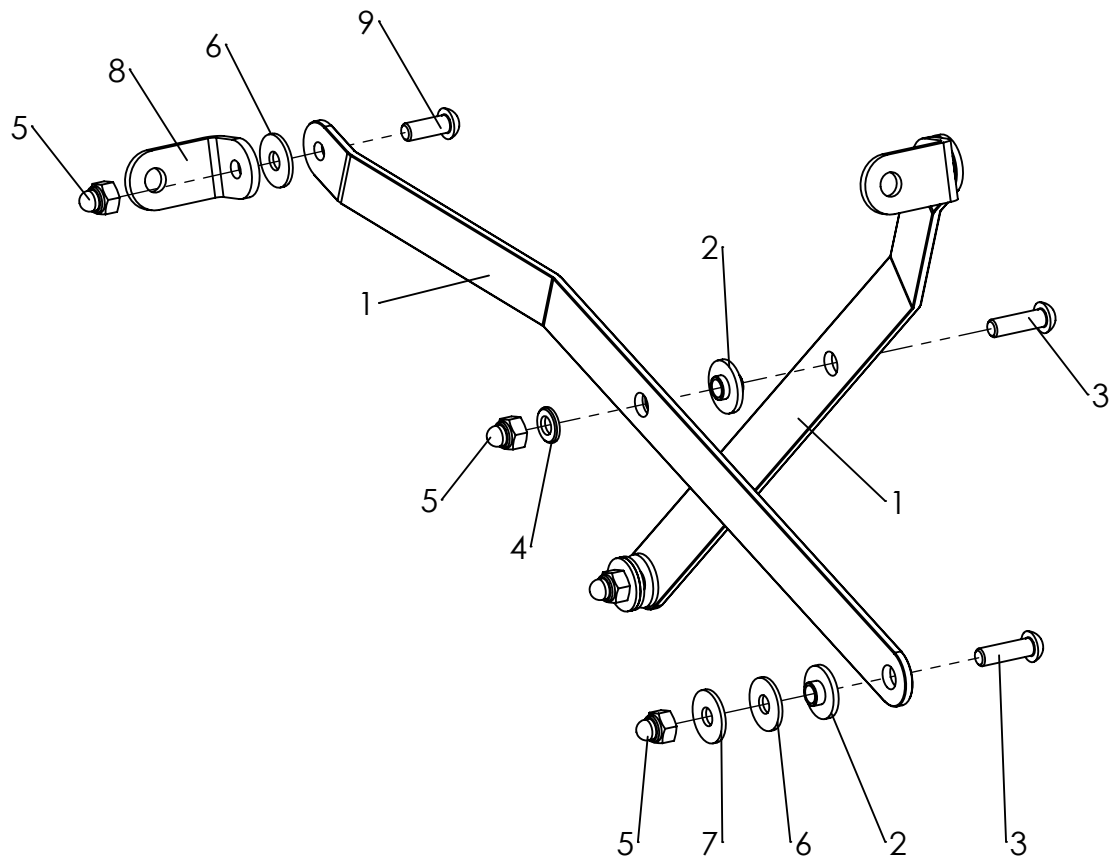

Pos	Article number	Articel description	Quantity	Unit
1-8	3210-1240E-015	Spread strud size 1, complete	1	Pc.
	3213-1240E-015	Spread strud size 2, complete	1	Pc.
1	3210-1241-015	Brace size 1, black	2	Pc.
	3213-1241-015	Brace size 2, black	2	Pc.
2	3210-1238	Shoulder bushing	3	Pc.
3	1029-0620	Button head socket screw M6x20	2	Pc.
4	1051-0006	Washer A6,4	2	Pc.
5	1044-0006	Hexagonal domed cap nut self-locking M6	5	Pc.
6	1113-0010	Washer PA black 18x6,4x1,5	4	Pc.
7	1055-0006	Washer zp A6,4	2	Pc.
8	3210-1715	joint angle	2	Pc.


Spare parts

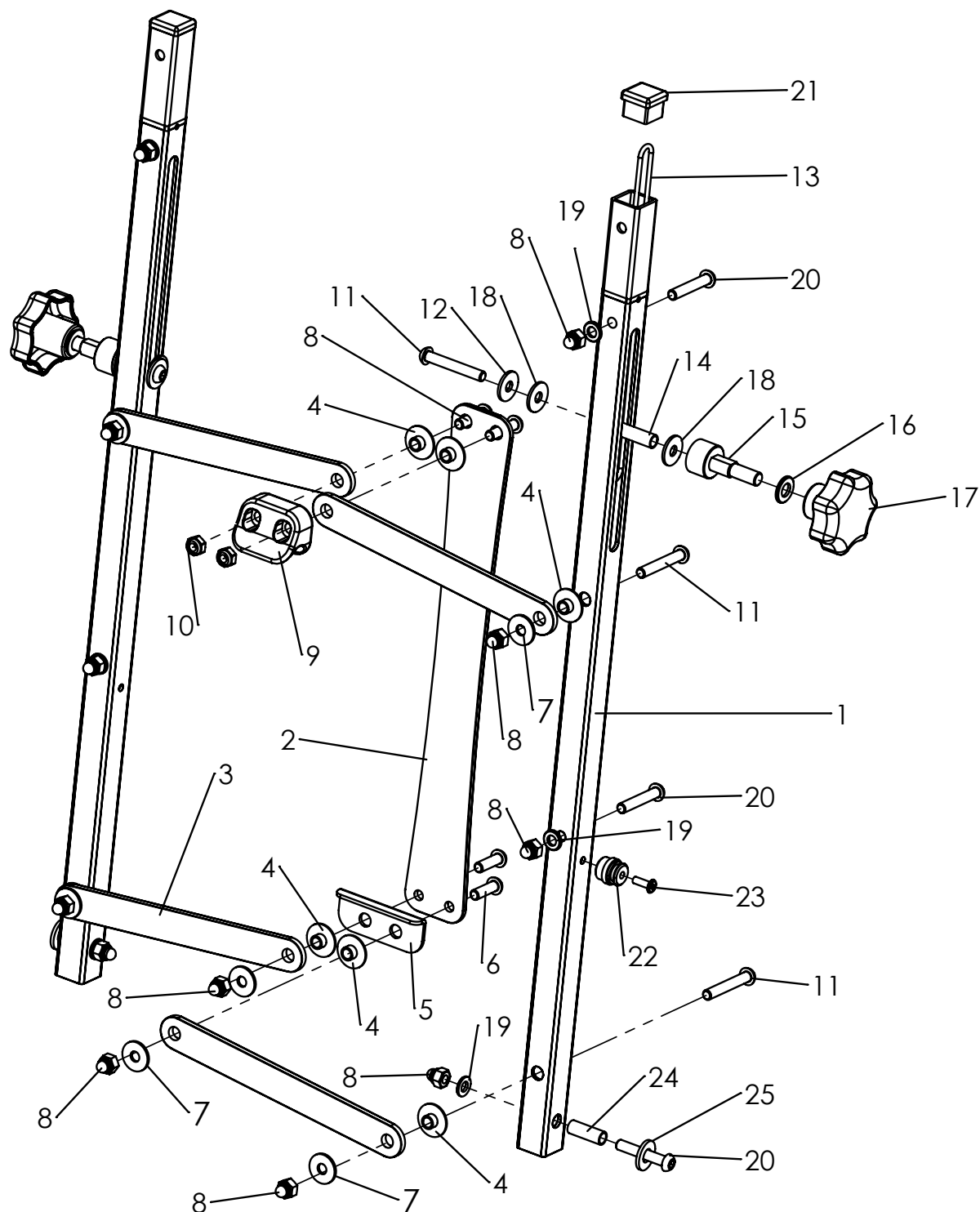

Spare parts

Pos	Article number	Articel description	Quantity	Unit
1-24	3210-1800E-052	Back rest frame size 1, back height 60 cm, black, complete	1	Pc.
	3213-1800E-052	Back rest frame size 2, complete, black	1	Pc.
1	3210-1811-052	Back tube standard, black matt	2	Pc.
	3210-1812-052	Back tube long, black matt	2	Pc.
2	3210-1814-052	Middle brace rear	1	Pc.
3	3210-1815-052	Locking lever size 1, black	4	Pc.
	3213-1815-052	Locking lever size 2, black	4	Pc.
4	3210-1830	Bearing bush	8	Pc.
5	3210-1816-015	Lock, FS black	1	Pc.
6	1029-0620	Button head socket screw M6x20	2	Pc.
7	1113-0010	Washer PA black DIN/ISO equal 9021 18x6,4x1,5	6	Pc.
8	1044-0006	Hexagonal domed cap nut self-locking M6	2	Pc.
9	3210-1817	Locking handle	1	Pc.
10	1042-0006	Hexagon nut self-locking M6	2	Pc.
11	1029-0640	button head socket screw M6x40	6	Pc.
12	1055-0006	Washer A6,4	2	Pc.
13	1150-0010	O-Ring 130x2,5	2	Pc.
14	1114-0025	Bushing zp 8x0,8x27	2	Pc.
15	3210-1850	Guide shaft	2	Pc.
16	1051-0008	Washer A8,4	2	Pc.
17	1122-5008	Star grip nut PA black M8	2	Pc.
18	1113-0064	Washer POM natural 8,4x18,0x2,0 mm	4	Pc.
19	1051-0006	Washer A6,4	6	Pc.
20	1029-0635	button head socket screw M6x35	6	Pc.
21	1141-0020	Ribbed insert VL 20x20x0,8-3 black	4	Pc.
22	3210-1813	O-ring bearing	2	Pc.
23	1062-0412	Blind rivet F4x12	2	Pc.
24	1114-0012	Bushing zp 8x0,8x22,5	2	Pc.
25	1113-0035	Washer PA 18x8,4x2 black	2	Pc.

Spare parts

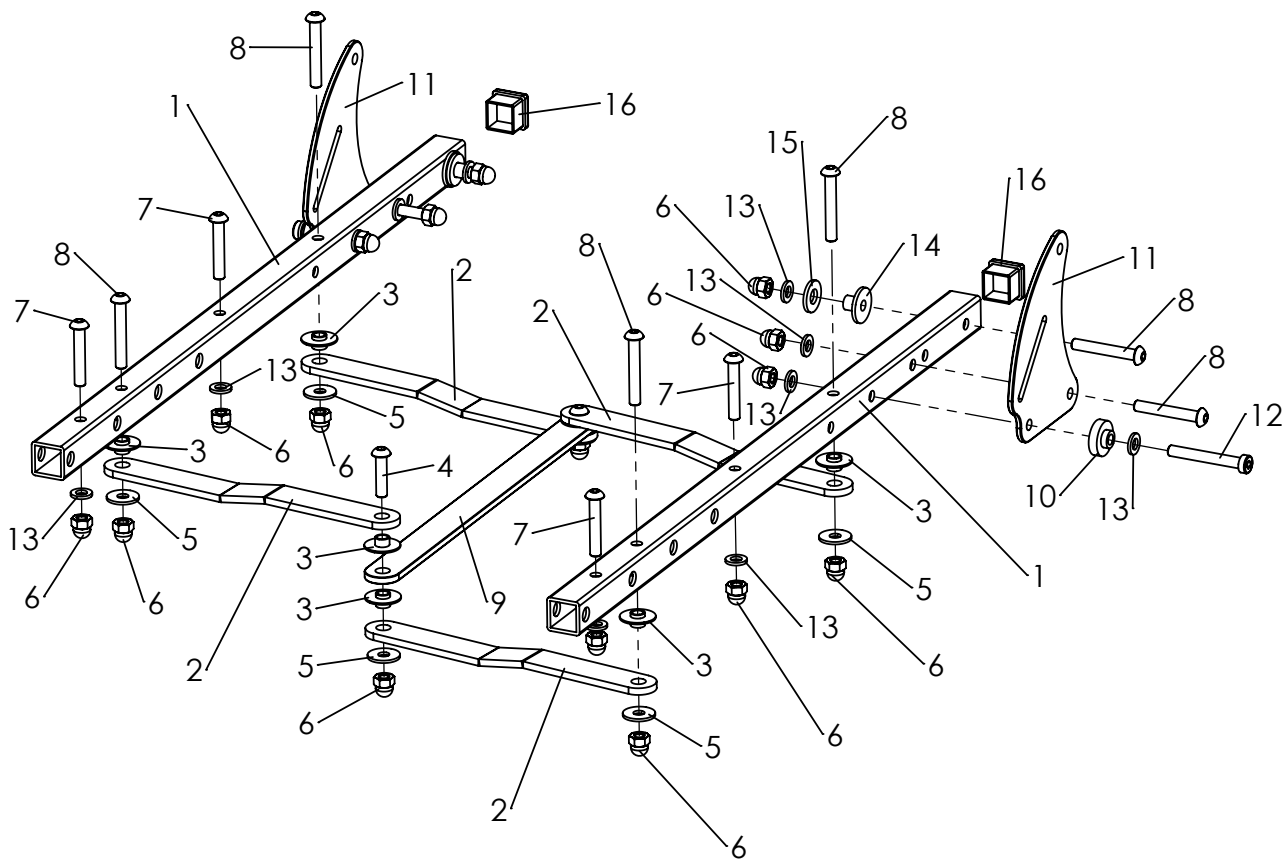


Pos	Article number	Articel description	Quantity	Unit
1-16	3210-1700E-052	Seat frame size 1 complete	1	Pc.
	3213-1700E-052	Seat frame complete size 2, black	1	Pc.
1	3210-1711	Seat tube, size 1	2	Pc.
	3213-1711	Seat tube, size 2	2	Pc.
2	3210-1740	Spreading lever size 1	1	Pc.
	3213-1740	Spreading lever size 2	1	Pc.
3	3210-1830	Bearing bush	8	Pc.
4	1029-0625	Button head socket screw M6x25	2	Pc.
5	1113-0010	Washer PA black 18x6,4x1,5	6	Pc.
6	1044-0006	Hexagonal domed cap nut self-locking M6	16	Pc.
7	1029-0635	Button head socket screw M6x35	4	Pc.
8	1029-0640	Button head socket screw M6x40	8	Pc.
9	3210-1730	Center strut	1	Pc.
10	3210-1712	Seat tube bearing	2	Pc.
11	3210-1713-015	Back stock size 1, black	2	Pc.
	3213-1713-015	Back stock size 2, black	2	Pc.
12	1022-0645	Hexagon socket head cap screw M6x45	2	Pc.
13	1051-0006	Washer A6,4	12	Pc.
14	3210-1714	Bearing bush	2	Pc.
15	1113-0035	Washer PA 18x8,4x2 black	2	Pc.
16	1141-0020	Ribbed insert VL 20x20x0,8-3 black	2	Pc.

Spare parts




Pos	Article number	Articel description	Quantity	Unit
1-5	3210-2101-000	ZIP reinforced nylon cover size1, back rest hight 60 cm	1	Pc.
	3210-2102-000	ZIP reinforced nylon cover size1, back rest hight 66 cm	1	Pc.
	3213-2101-000	ZIP reinforced nylon cover size 2, back rest hight 68 cm	1	Pc.
1	3210-2110-000	Seat cover size 1	1	Pc.
	3213-2110-000	Seat cover size 2	1	Pc.
2	3210-1771	Fastening adapter for seat cover size 1	2	Pc.
	3213-1771	Fastening adapter for seat cover size 2	2	Pc.
3	3210-2120-000	Side cover size 1	1	Pair
	3213-2120-000	Side cover size 2	1	Pair
4	3210-2130-000	Back cover standard, size 1	1	Pc.
	3210-2140-000	Back cover long	1	Pc.
	3213-2130-000	Back cover size 2	1	Pc.
5	3210-1861	Fastening adapter for back cover	2	Pc.
	3210-1871	Fastening adapter for back cover, long	2	Pc.

Spare parts

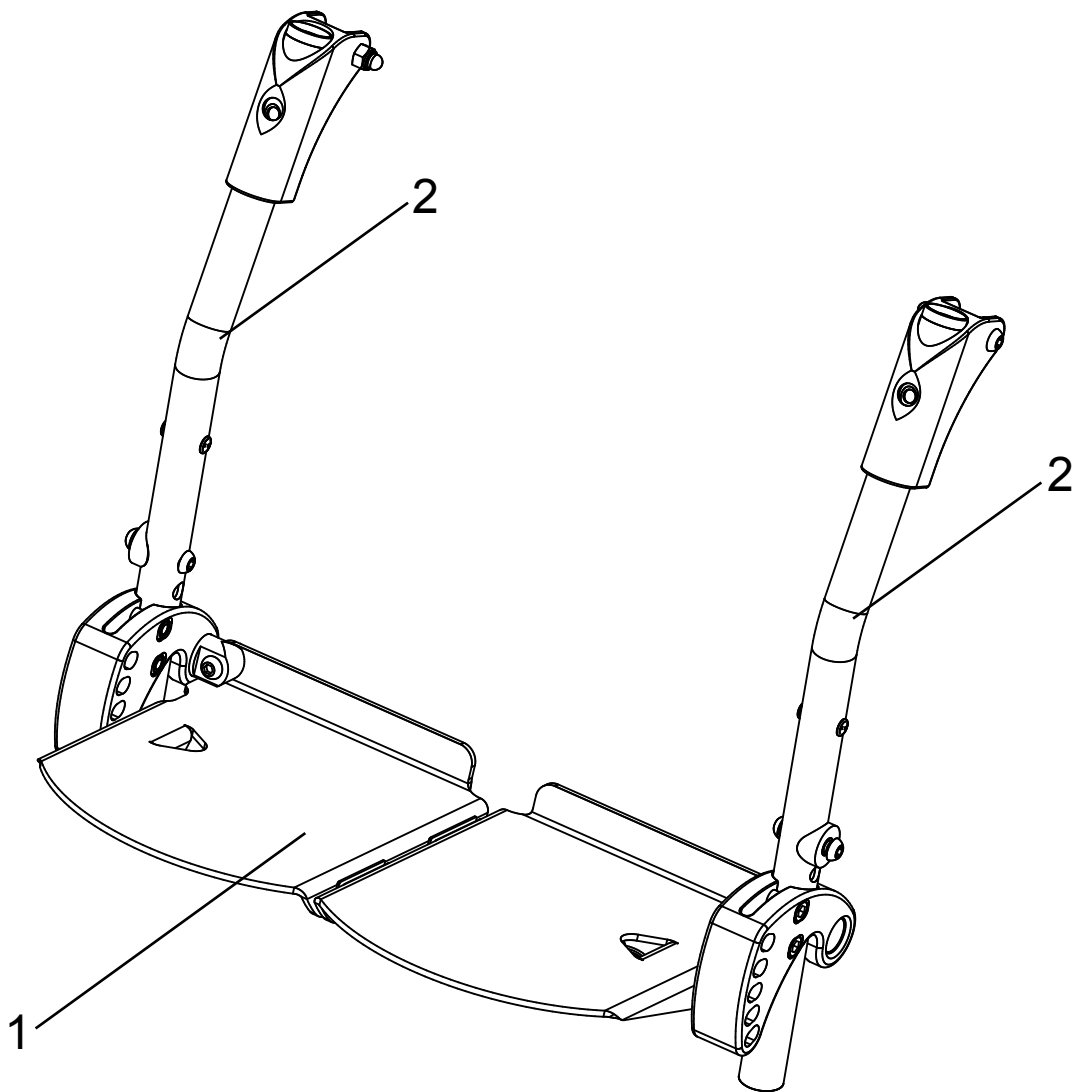


Pos	Article number	Articel description	Quantity	Unit
1	3210-2410-015	Foot board, size 1	1	Pc.
	3213-2410-015	Foot board, size 2	1	Pc.
2	3210-2320-052	Foot board reciver, size 1	2	Pc.
	3213-2320-052	Foot board reciver, size 2	2	Pc.

Spare parts

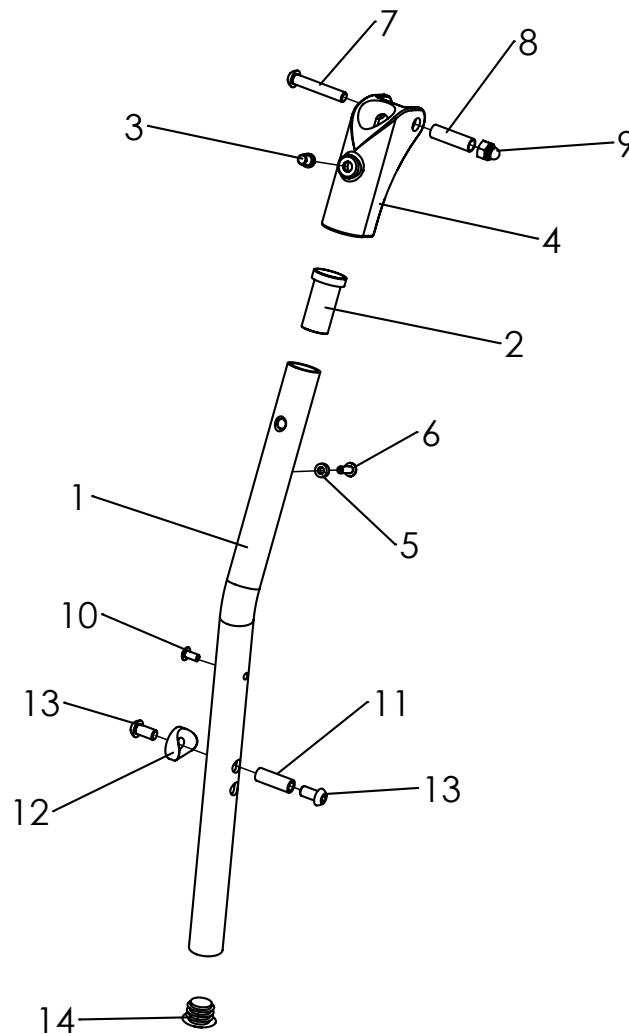


Pos	Article number	Articel description	Quantity	Unit
1-14	3210-2320E-052	Foot board receiver size 1, black	1	Pc.
	3213-2320E-052	Foot board receiver size 2, schwarz	1	Pc.
1	3210-2321-052	Foot rest receiver size 1, black	1	Pc.
	3213-2321-052	Foot rest receiver size 2, black	1	Pc.
2	3210-2323	Guide sleeve	1	Pc.
3	3210-2324	Trigger	1	Pc.
4	3210-2322	Swivel guide	1	Pc.
5	1112-0010	Spacer PA black 8x4,3x3	1	Pc.
6	1062-0412	Blind rivet F4x12	1	Pc.
7	1029-0635	Button head socket screw M6x35	1	Pc.
8	1114-0025	Bushing zp 8x0,8x27	1	Pc.
9	1044-0006	Hexagonal domed cap nut self-locking M6	1	Pc.
10	1062-0408	Blind rivet F4x8	1	Pc.
11	1114-0058	Threaded bushing 8xM6x27,5	1	Pc.
12	1111-0018	Saddle washer PA black 20x8.4x7	1	Pc.
13	1029-0612	Button head socket screw M6x12	2	Pc.
14	1141-0006	Ribbed insert SFL 20x0,8-2,5	1	Pc.

Spare parts

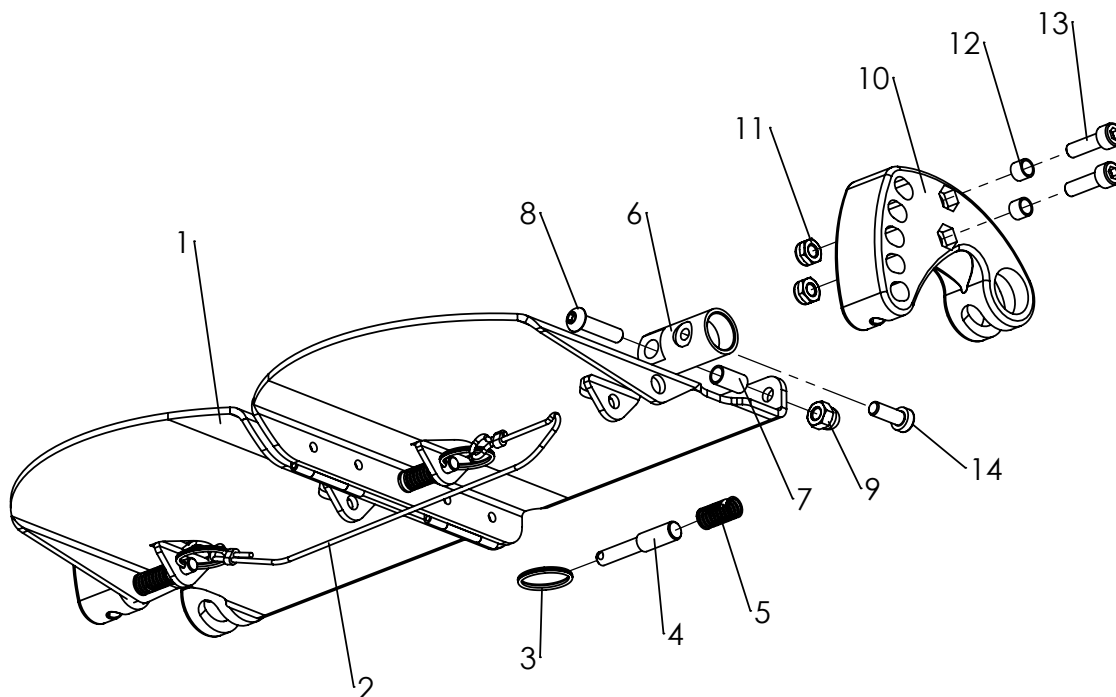


Pos	Article number	Articel description	Quantity	Unit
1-14	3210-2410E-015	Foot board assembly, complete size 1	1	Pc.
	3213-2410E-015	Foot board assembly, complete size 2	1	Pc.
1-5	3210-2330E-015	Foot board with locking mechanism, size 1	1	Pc.
	3213-2330E-015	Foot board with locking mechanism, size 2	1	Pc.
1	3210-2330-015	Foot board pair assembly, size 1, black	1	Pc.
	3213-2330-015	Foot board pair assembly, size 2, black	1	Pc.
2	3210-2439	Release cable for footplate locking size 1	1	Pc.
	3213-2439	Release cable for footplate locking size 2	1	Pc.
3	1129-0003	Key ring 24x1,5	3	Pc.
4	3210-2335	Lock pin	3	Pc.
5	1131-0036	Compression spring D-127	3	Pc.
6	3210-2333	Rotation bearing (L-R is the same)	2	Pc.
	3213-2332	Rotation bearing left size 2	1	Pc.
	3213-2333	Rotation bearing right size 2	1	Pc.
7	1114-0018	Bushing zp 8x0,8x14,5	2	Pc.
8	1029-0625	Button head socket screw M6x25	2	Pc.
9	1044-0006	Hexagonal domed cap nut self-locking M6	2	Pc.
10	3210-2331	Clamp holder	2	Pc.
11	1042-0006	Hexagon nut self-locking M6	4	Pc.
12	1114-0010	Bushing zp 8x0,8x6,0-0,2	4	Pc.
13	1021-0620	Hexagon socket head cap screw M6x20	4	Pc.
14	1022-0616	Hexagon socket head cap screw M6x16	2	Pc.

Spare parts

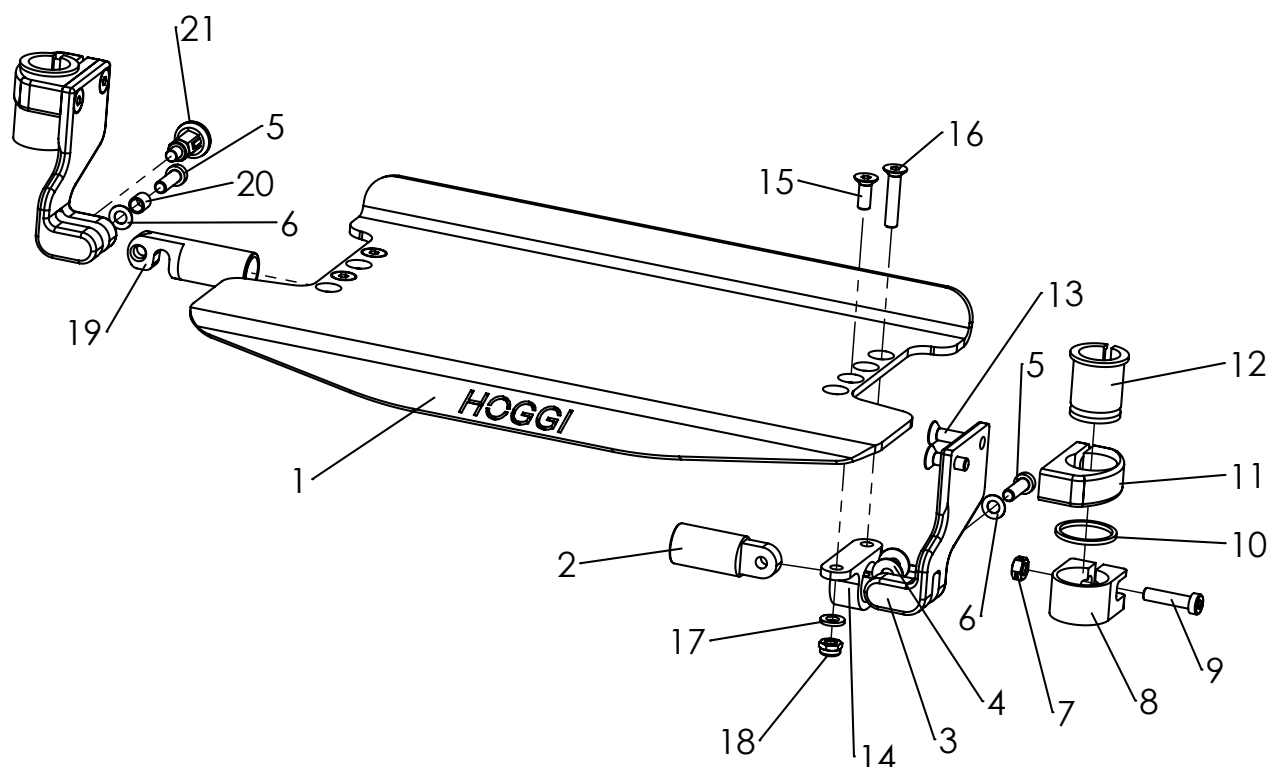


Pos	Article number	Articel description	Quantity	Unit
1-21	3210-2510E-015	Footboard system continuous, angle adj., foldable, swing away size ' 1	1	Pc.
	3213-2510E015	Footboard system continuous, angle adj., foldable, swing away size :	1	Pc.
1	3210-2530-015	Footboard throughout ZIP Gr.1	1	Pc.
	3213-2530-015	Footboard throughout ZIP Gr.2	1	Pc.
2	3261-2713	Joint shaft	1	Pc.
3	3210-2535-007	Footboard holder ZIP left black anodised	1	Pc.
	3210-2534-007	Footboard holder ZIP right black anodised	1	Pc.
4	1113-0046	Washer PE white 16x6,4x1	2	Pc.
5	1022-0620	Hexagon socket head cap screw M6x20	3	Pc.
6	1052-0006	Spring washer B6	2	Pc.
7	1049-0068	BILOC-nut M6	2	Pc.
8	3210-2531-007	Clamp footrest ZIP black anodized	2	Pc.
9	1022-0625	Hexagon socket head cap screw M6x25	2	Pc.
10	1113-0067	Washer PA natural d26xD30,0x2,0	2	Pc.
11	3210-2533-007	Swivel bracket footrest ZIP black anodized	2	Pc.
12	3210-2532	Sliding sleeve footrest ZIP	2	Pc.
13	1024-0620	Hexagon socket countersunk head screw M6x20	4	Pc.
14	3261-2116-006	Footboard clamp anodized	2	Pc.
15	1024-0616	Hexagon socket countersunk head screw M6x16	2	Pc.
16	1024-0630	Hexagon socket countersunk head screw M6x30	2	Pc.
17	1051-0006	Washer A6,4	2	Pc.
18	1042-0006	Hexagon nut self-locking M6	2	Pc.
19	3210-2536-007	Latch Footrest, black anodized	1	Pc.
20	1114-0055	Bushing 8x0,8x8,7	1	Pc.
21	1127-0011	Lock pin ZS-M10x1-KS with lock	1	Pc.

Spare parts

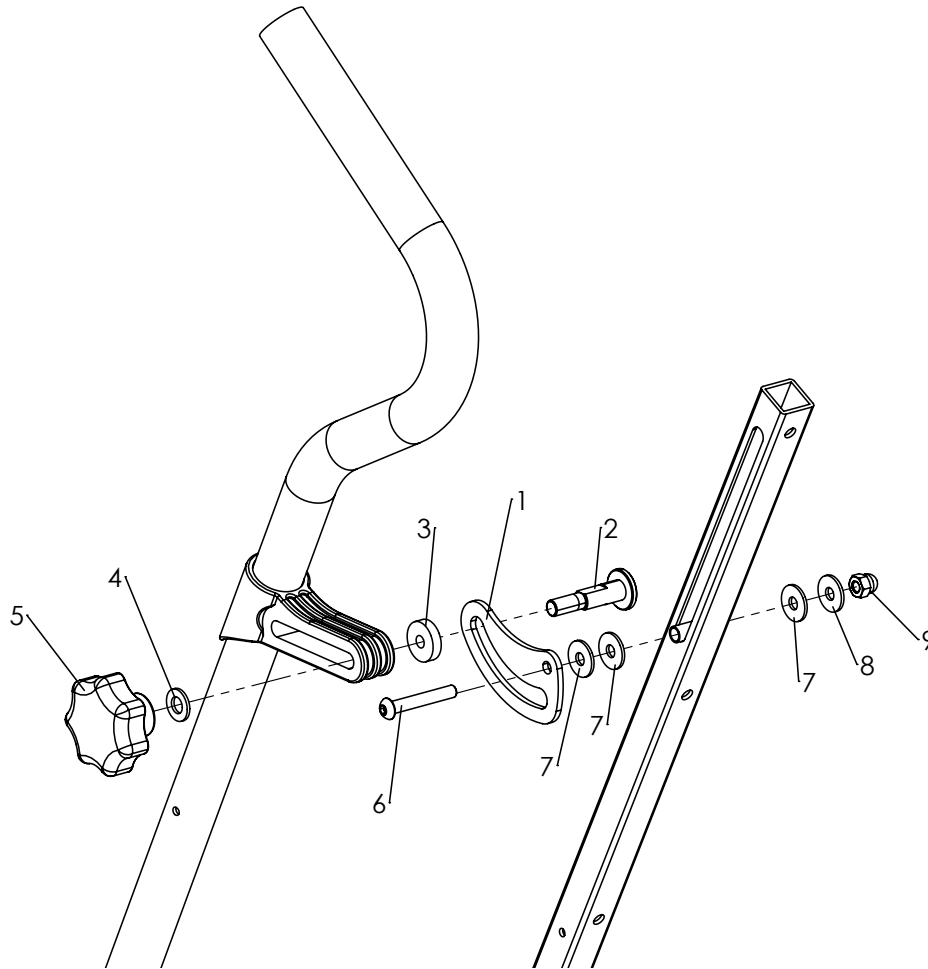


Pos	Article number	Articel description	Quantity	Unit
1-9	3210-8500E	Back rest angle extention up to 5°	1	Set
1	3210-1851-015	Angle plate black	1	Pc.
2	3210-1852	Guiding shaft	1	Pc.
3	3210-1853	Spacer ring	1	Pc.
4	1051-0008	Washer A8,4	1	Pc.
5	1112-5008	Star grip nutt	1	Pc.
6	1029-0640	button head socket screw M6x40	1	Pc.
7	1113-0064	Washer POM natural 8,4x18,0x2,0 mm	3	Pc.
8	1055-0006	Washer A6,4	1	Pc.
9	1044-0006	Hexagonal domed cap nut self-locking M6	1	Pc.

Spare parts

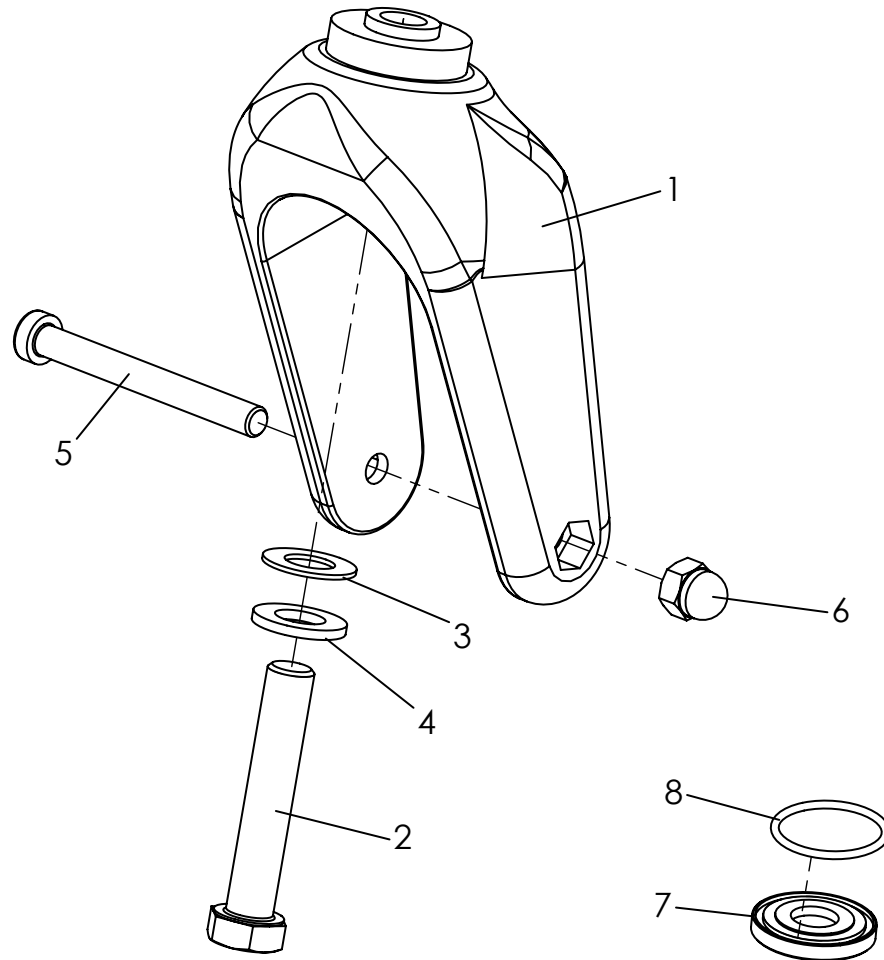


Pos	Article number	Articel description	Quantity	Unit
1-6	1281-0007E	Castor fork with fasteners	1	Pc.
1	1281-0007	Castor fork LG-K-200-12 schw 200x60	1	Pc.
2	1012-1265	Hexagon head screw M12x65	1	Pc.
3	1051-0012	Washer 125 A	1	Pc.
4	1052-0012	Spring washer B12	1	Pc.
5	1022-0875	Hexagon socket head cap screw M8x75	1	Pc.
6	1044-0008	Hexagonal domed cap nut self-locking M8	1	Pc.
7-8	3202-7120E	Sealing for steering fork bearing for two forks	1	Set
7	3224-1174	Sealing disc caster fork	2	Pc.
8	1150-0018	O-ring 26x2	2	Pc.

Spare parts

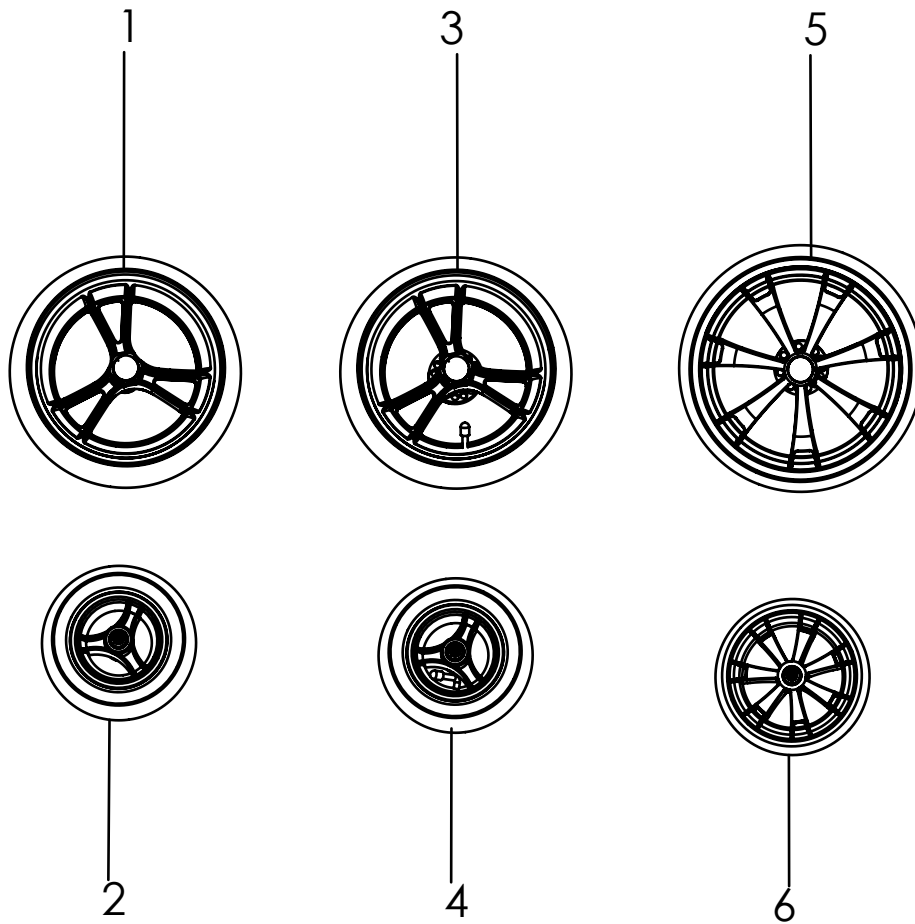



Pos	Article number	Articel description	Quantity	Unit
1	1221-2961	PU-wheel 290x60 with break disk and quick release	2	Pc.
2	1221-1950	PU-wheel 190x50	2	Pc.
3	1211-2961	Pneumatic wheel 290x60, quick release	2	Pc.
4	1211-1950	Pneumatic wheel 190/50	2	Pc.
5	1221-3061	PU-Wheel R300/60, chrome, with break disk	2	Pc.
	1221-3063	PU-Wheel R300/60, ABS black, with break disk	2	Pc.
6	1221-2048	PU-Wheel R200/48 chrome rim	2	Pc.
	1221-2049	PU-Wheel R200/48 ABS black	2	Pc.

Spare parts

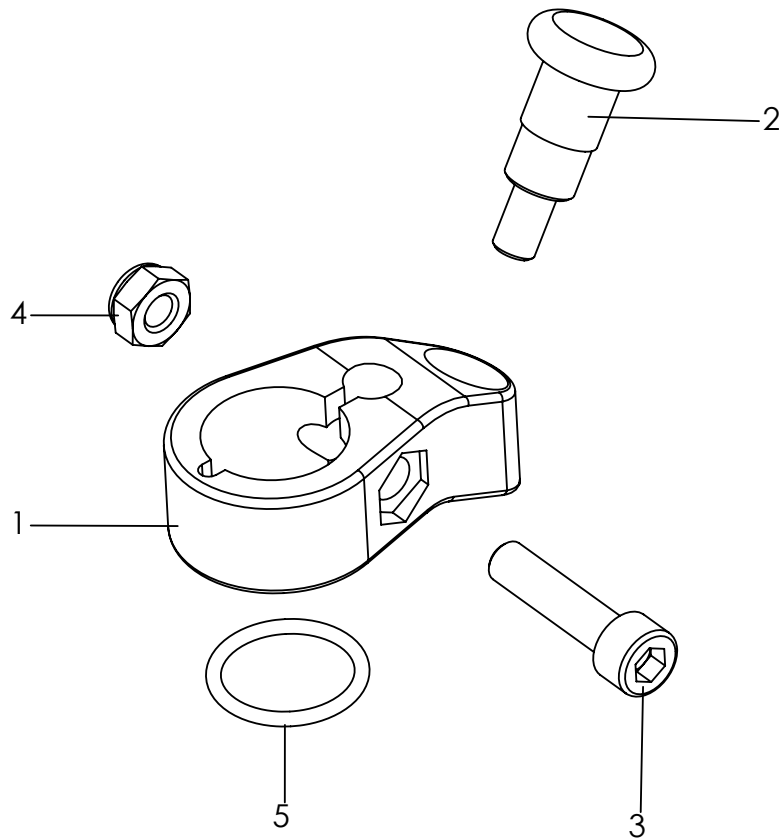


Pos	Article number	Articel description	Quantity	Unit
1-5	3202-7100	Swivel lock, for two castor fork`s	1	Set
1	3202-7110	Swivel lock housing	2	Pc.
2	1127-0009	Lock pin, with interlocksystem	2	Pc.
3	1021-0625	Hexagon socket head cap screw M6x25	2	Pc.
4	1042-0006	Hexagon nut self-locking M6	2	Pc.
5	1150-0001	O-ring NBR 18x2	2	Pc.

Spare parts

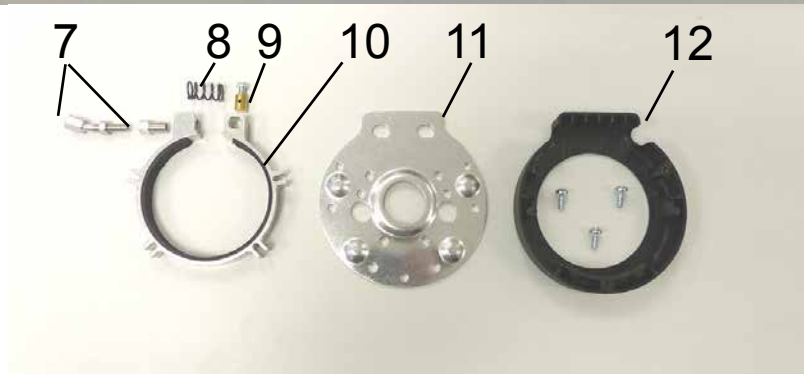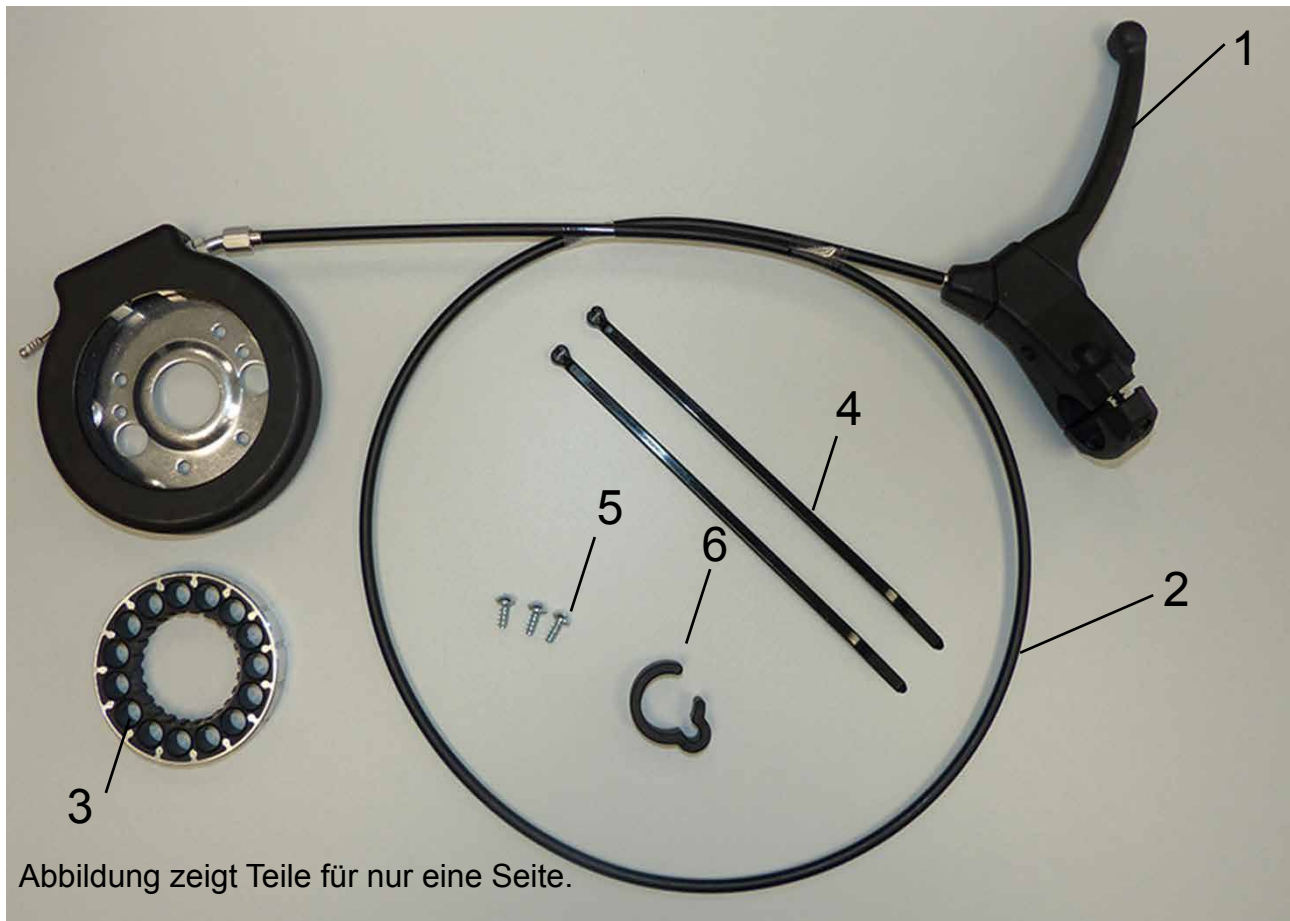

Pos	Article number	Articel description	Quantity	Unit
1-6	3210-7201	Attendant brake, two hand use	1	Set
1	1209-0020	Hand brake 36° no lock 25 mm	2	Pc.
2	1209-0001	Bowden wire set 1,5m	2	Pc.
3	1204-0005	Brake disc with aluminium-ring	2	Pc.
4	1089-0004	Cable tie, black, 5x180	4	Pc.
5	1039-0001	screw for plastic, cross rcessed pan head w. flange 4x10	6	Pc.
6	1149-0001	Cable clip D 25	2	Pc.
Small parts				
7	3231-7217	Setscrew for cable, 160° angle	2	Pc.
8	1131-0073	Compression spring	2	Pc.
9	1099-0015	Screw nipple	2	Pc.
10	3231-7213	Alu-profile bracket with brake pad	2	Pc.
11	3231-7214	Base plate for hub brake	2	Pc.
12	3231-7212	Cover for hub brake	2	Pc.

Spare parts

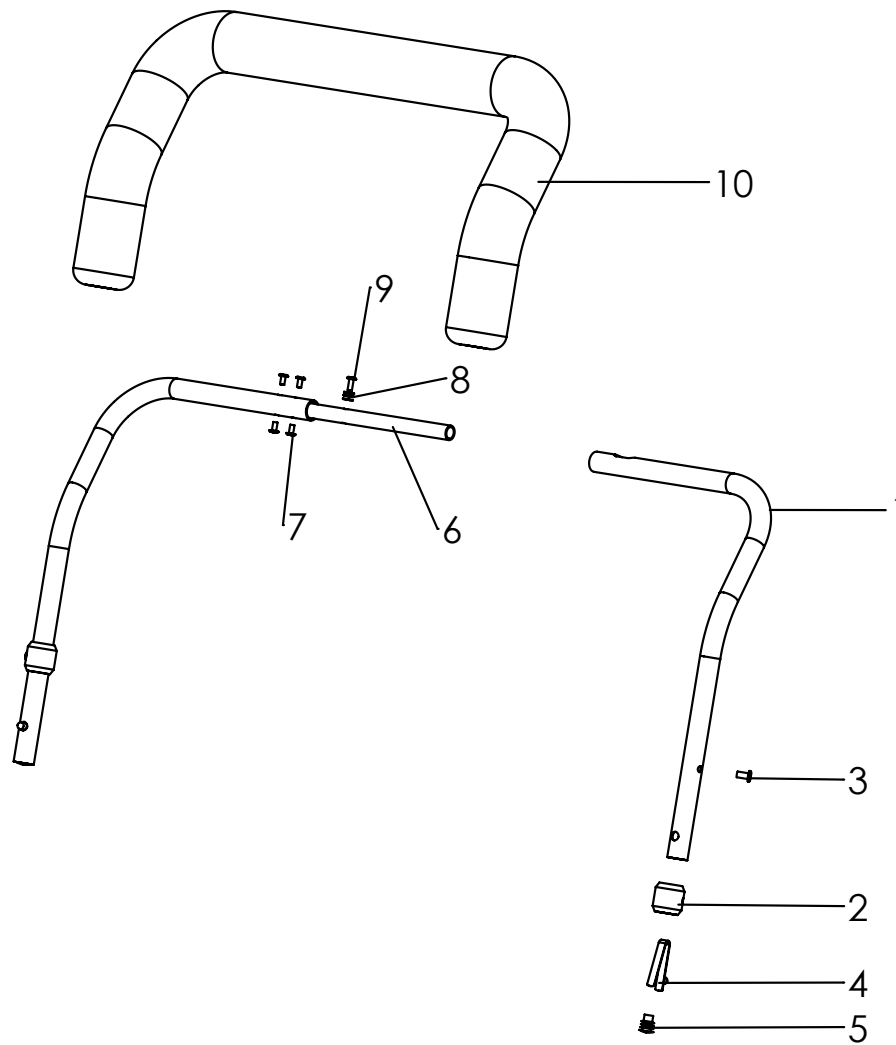


Pos	Article number	Articel description	Quantity	Unit
1-10	3210-8100-xxx	Grab rail with upholstery size 1	1	Set
	3213-8100-xxx	Grab rail with upholstery size 2	1	Pc.
1	3210-8111-002	Grab rail tube left size 1	1	Pc.
	3210-8112-002	Grab rail tube right size 1	1	Pc.
	3213-8111-002	Grab rail tube left size 2	1	Pc.
	3213-8112-002	Grab rail tube right size 2	1	Pc.
2	3201-8120	Bushing	2	Pc.
3	1062-0408	Blind rivet F4x8	2	Pc.
4	1134-0005	Snap button A 102	2	Pc.
5	1141-0010	Ribbed insert SFL 10x0,8-1.5	2	Pc.
6	3210-8114	Connecting pipe	1	Pc.
7	1062-0306	Blind rivet F3x6	4	Pc.
8	1051-0003	Washer A3,2	3	Pc.
9	1062-0308	Blind rivet F3x8	1	Pc.
10	7000-0038	PE foam profile, size 1	1	Pc.
		PE foam profile, size 2	1	Pc.

66 cm


70 cm

Spare parts

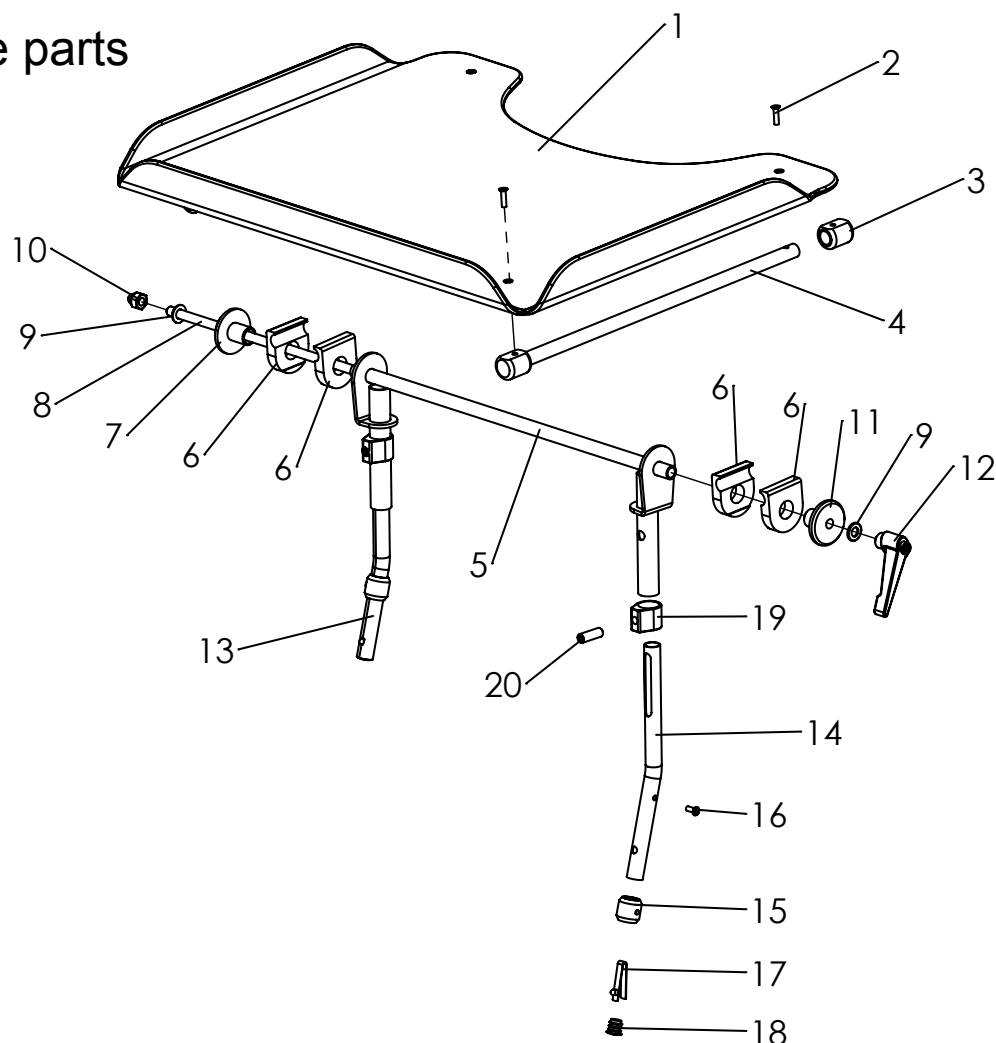


Pos	Article number	Articel description	Quantity	Unit
1-20	3210-8202	Therapy tray size 1, transparent, height adjustable	1	Set
	3210-8203	Therapy tray size 1, grey, height adjustable	1	Set
	3213-8202	Therapy tray size 2, transparent, height adjustable	1	Set
	3213-8203	Therapy tray size 2, grey, height adjustable	1	Set
1	3210-8290	Tray plate clear	1	Pc.
	3210-8291	Tray plate grey	1	Pc.
2	1063-0416	Blind rivet 4x16	4	Pc.
3	3201-8280	Bearing block	4	Pc.
4	3201-8270	Rail	2	Pc.
5	3210-8220	Basic support height adjustable size 1	1	Pc.
	3213-8220	Basic support height adjustable size 2	1	Pc.
6	3201-8230	Clamp half	4	Pc.
7	3201-8250	Clamp disk M8	1	Pc.
8	1039-0002	Thread rod M8x1000 (shorten to 43,5cm)	1	Pc.
9	1051-0008	Washer A8,4	2	Pc.
10	1044-0008	Hexagonal domed cap nut self-locking M8	1	Pc.
11	3201-8240	Clamp disk D 8,5	1	Pc.
12	1121-0800	Locking lever PA black M8	1	Pc.
13	3210-8215	Support hight adjustable left	1	Pc.
14	3210-8216	Support hight adjustable left	1	Pc.
15	3201-8120	Bushing	2	Pc.
16	1062-0410	Blind rivet F4x10	2	Pc.
17	1134-0005	Snap button A 102	2	Pc.
18	1141-0001	Ribbed insert SFL 14x0,8-2	2	Pc.
19	3201-8221	Clamp ring for height adjustable tray	2	Pc.
20	1031-0825	Hexagon socket set screw M8x25	2	Pc.

Spare parts

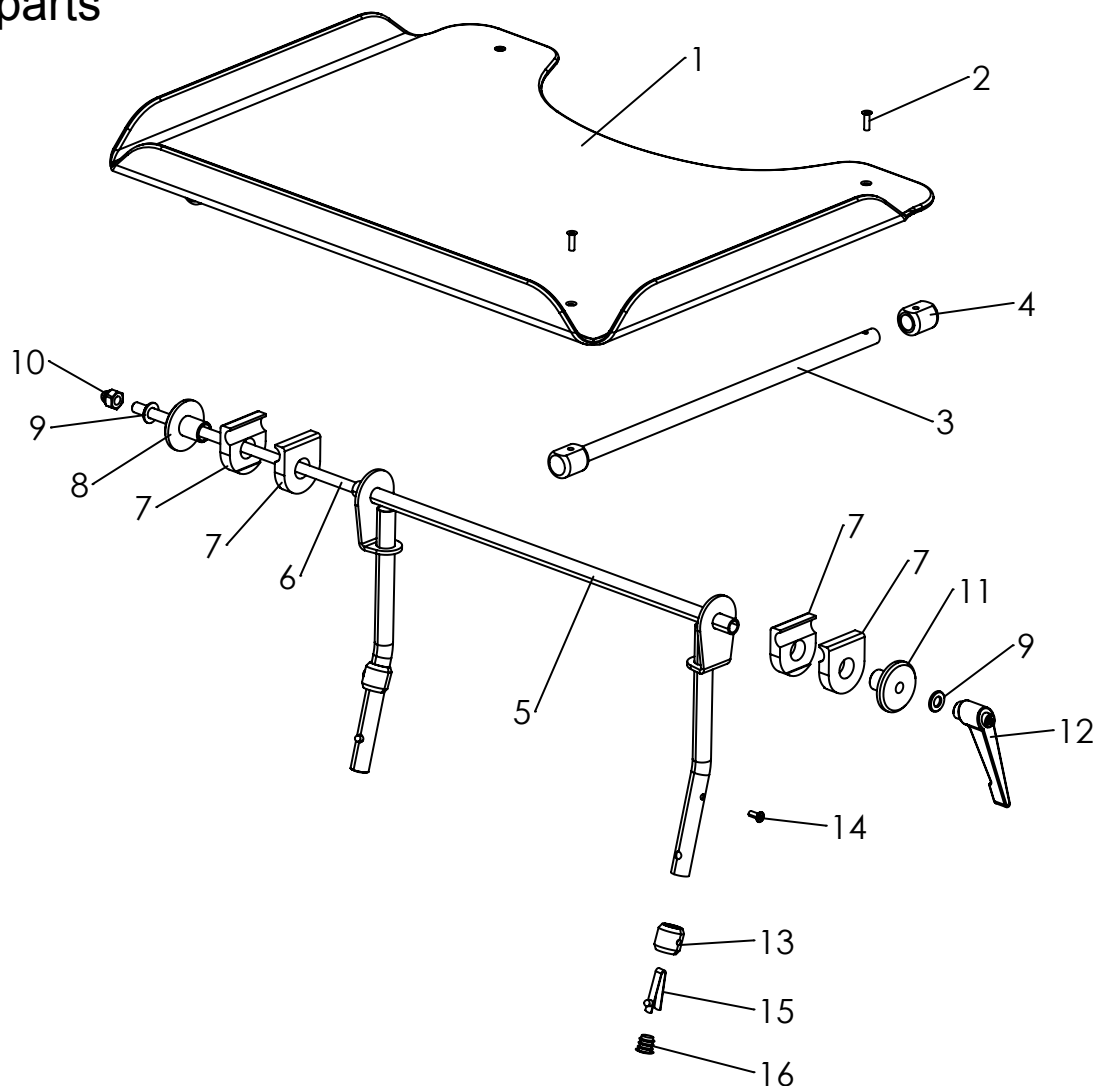


Pos	Article number	Articel description	Quantity	Unit
1-16	3210-8200	Therapy tray size 1, transparent	1	Set
	3210-8201	Therapy tray size 1, grey	1	Set
	3213-8200	Therapy tray size 2, transparent	1	Set
	3213-8201	Therapy tray size 2, grey	1	Set
1	3210-8290	Tray plate clear	1	Pc.
	3210-8291	Tray plate grey	1	Pc.
2	1063-0416	Blind rivet 4x16	4	Pc.
3	3201-8270	Rail	2	Pc.
4	3201-8280	Bearing block	4	Pc.
5	3210-8210	Basic support size 1	1	Pc.
	3213-8210	Basic support size 2	1	Pc.
6	1039-0002	Thread rod M8x1000 <i>(shorten to 43,5cm)</i>	1	Pc.
7	3201-8230	Clamp half	4	Pc.
8	3201-8250	Clamp disk M8	1	Pc.
9	1051-0008	Washer A8,4	2	Pc.
10	1044-0008	Hexagonal domed cap nut self-locking M8	1	Pc.
11	3201-8240	Clamp disk D 8,5	1	Pc.
12	1121-0800	Locking lever PA black M8	1	Pc.
13	3201-8120	Bushing	2	Pc.
14	1062-0410	Blind rivet F4x10	2	Pc.
15	1134-0005	Snap button A 102	2	Pc.
16	1141-0001	Ribbed insert SFL 14x0,8-2	2	Pc.

Spare parts

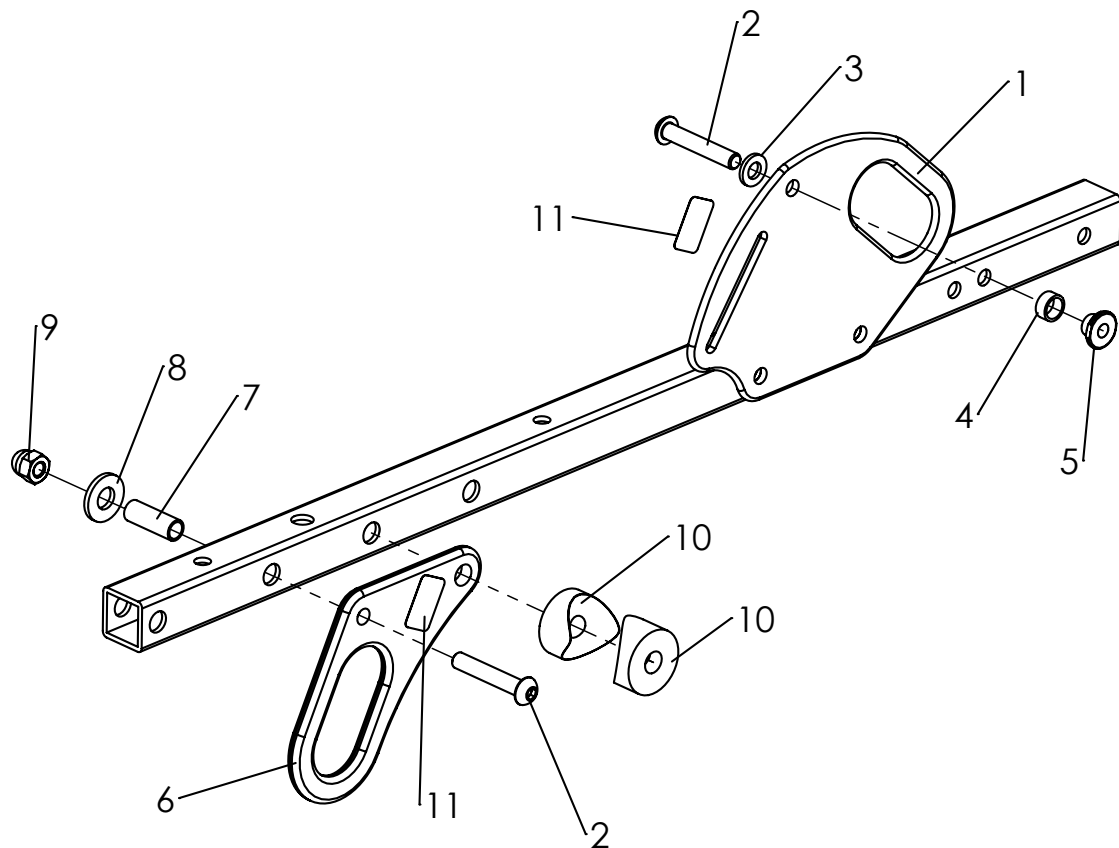


Pos	Article number	Articel description	Quantity	Unit
1-12	3210-7300E	Tie down kit retrofit, size 1	1	Set
1	3210-7330-002	Back bearing with eye, silver	2	Pc.
2	1029-0635	Button head socket screw M6x35	4	Pc.
3	1051-0006	Washer A6,4	2	Pc.
4	3201-7350	Slide bushing	2	Pc.
5	3201-7360	Belt safety knob	2	Pc.
6	3210-7320-002	Front tie down bracket, silver	2	Pc.
7	1114-0012	Bushing zp 8x0,8x22,5	2	Pc.
8	1113-0035	Washer PA 18x8,4x2 black	2	Pc.
9	1044-0006	Hexagonal domed cap nut self-locking M6	2	Pc.
10	1111-0014	Saddle washer PA black 24x8.4x6	4	Pc.
11	1914-0007	Adhesive lable with hook symbol (4 pcs.)	1	Pc.

Spare parts

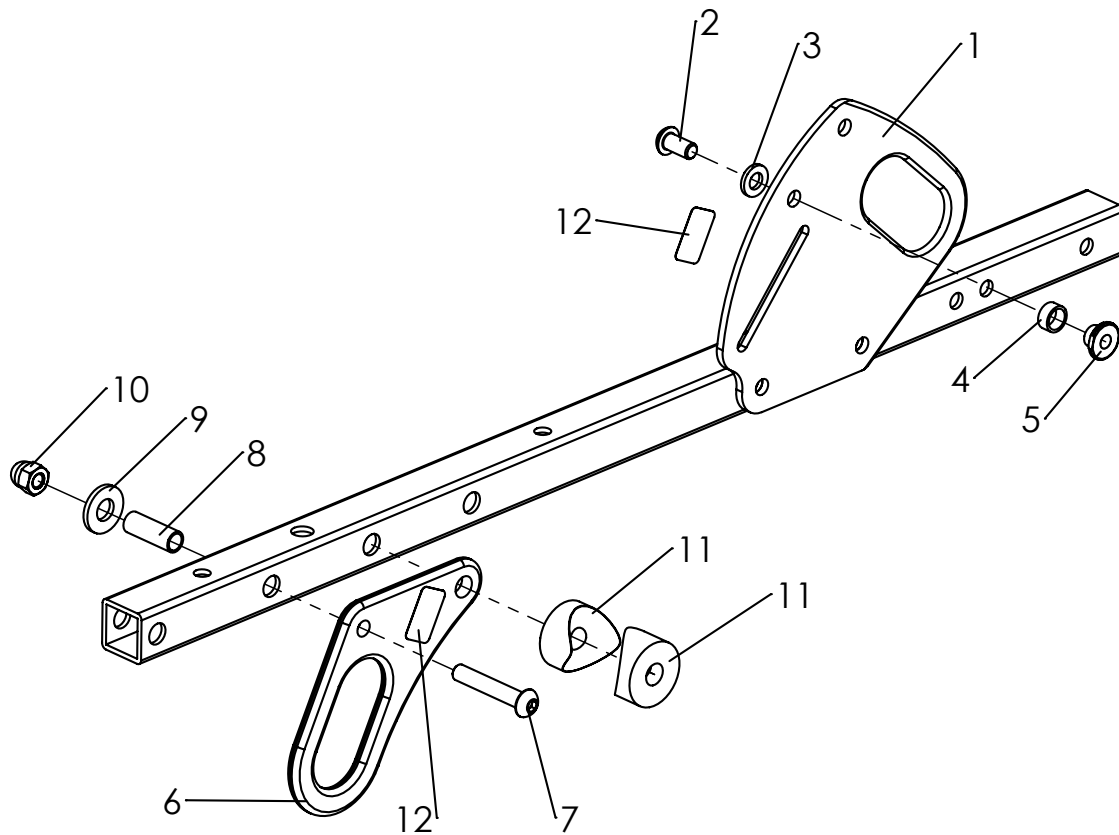

Pos	Article number	Articel description	Quantity	Unit
1-12	3213-7300E	Tie down kit retrofit, size 2	1	Set
1	3213-7330-002	Back bearing with eye, silver size 2	2	Pc.
2	1029-0612	Button head socket screw M6x12	4	Pc.
3	1051-0006	Washer A6,4	2	Pc.
4	3201-7350	Slide bushing	2	Pc.
5	3201-7360	Belt safety knob	2	Pc.
6	3210-7320-002	Front tie down bracket, silver	2	Pc.
7	1029-0635	button head socket screw M6x35	2	Pc.
8	1114-0012	Bushing zp 8x0,8x22,5	2	Pc.
9	1113-0035	Washer PA 18x8,4x2 black	2	Pc.
10	1044-0006	Hexagonal domed cap nut self-locking M6	2	Pc.
11	1111-0014	Saddle washer PA black 24x8.4x6	4	Pc.
12	1914-0007	Adhesive lable with hook symbol (4 pcs.)	1	Pc.


Spare parts

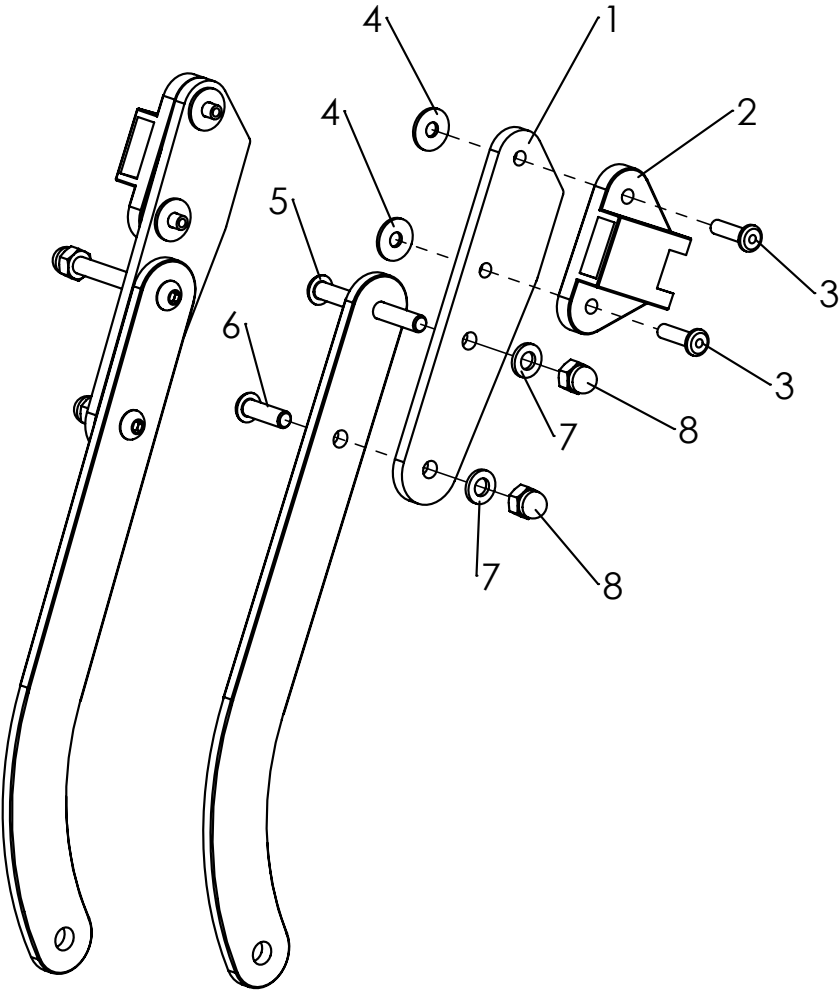

Spare parts

Pos	Article number	Articel description	Quantity	Unit
1-12	3210-9300-xxx	Canopy size 1, incl. adapter and rain protection, grey	1	Set
	3213-9300-xxx	Canopy size 2, incl. adapter and rain protection, grey	1	Set
1-8	3210-9330E	ZIP size 1 canopy adapter compl. (retrofit)	1	Pair
	3213-9330E	ZIP canopy adapter size 2 compl. (retrofit)	1	Pair
1	3210-9311	Mounting adapter size 1	2	Pc.
	3213-9311	Mounting adapter size 2	2	Pc.
2	7000-0005	Canopy holder	2	Pc.
3	1062-0518	Blind rivet F5x18	4	Pc.
4	1055-0005	Washer A5,3	4	Pc.
5	1029-0655	Button head socket screw M6x55	2	Pc.
6	1029-0620	Button head socket screw M6x20	2	Pc.
7	1051-0006	Washer A6,4	4	Pc.
8	1044-0006	Hexagonal domed cap nut self-locking M6	4	Pc.
9	7000-0036-xxx	Canopy (w/o rain cover) Zip, grey	1	Pc.
10	7000-0053	Rain cover for canopy, ZIP size 1	1	Pc.
	7000-0037	Rain cover for canopy Zip size 2	1	Pc.
11	7000-0004	Ratchet joint for canopy with fasteners	2	Pc.

Spare parts

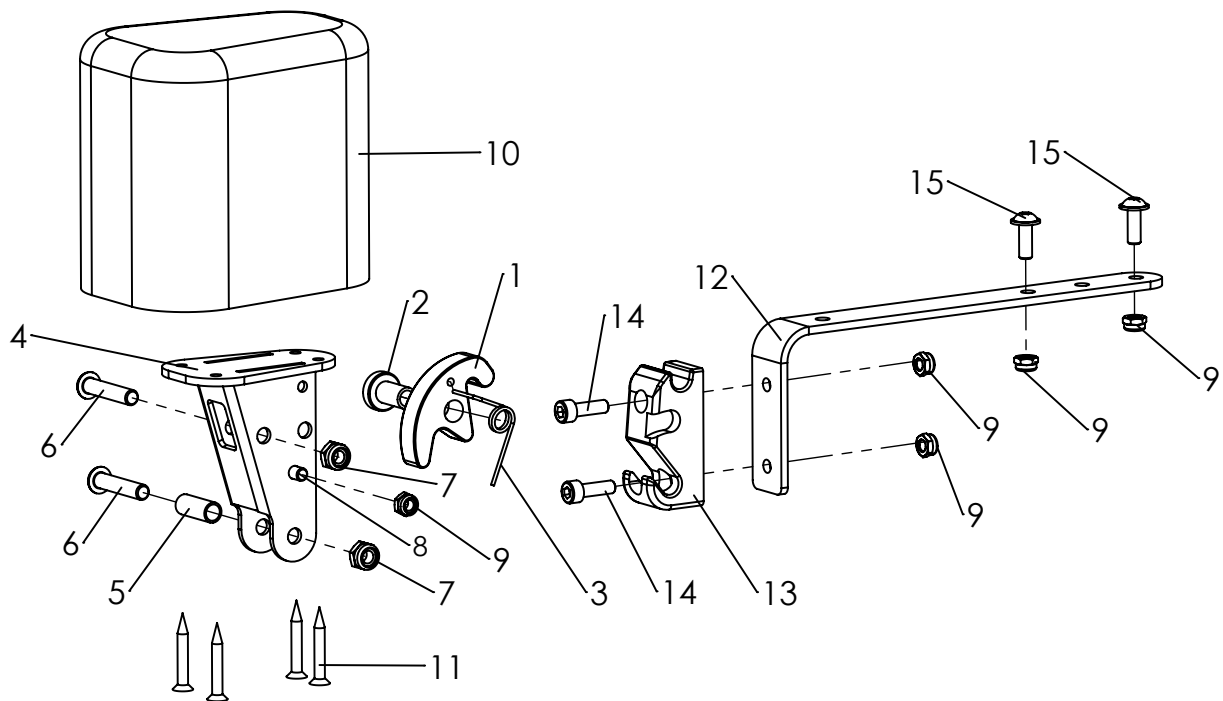

Pos	Article number	Articel description	Quantity	Unit
1-15	3210-7800	Abduction block size 2 swing away and detachable, w/o cover	1	Pc.
	3213-7800	Abduction block size 3 swing away and detachable, w/o cover	1	Pc.
1	3231-7841	Arresting clamp	1	Pc.
2	3231-7842	Bearing arresting clamp	1	Pc.
3	1133-0020	Leg spring 1,25 x 9,25 x 2,25	1	Pc.
4	3231-7820-032	A-wedge support	1	Pc.
5	1114-0013	Bushing zp 8x0,8x20	1	Pc.
6	1029-0630	Button head socket screw M6x30	2	Pc.
7	1042-0006	Hexagon nut self-locking M6	2	Pc.
8	1021-0508	Cylinder head screw M5x8	1	Pc.
9	1042-0005	Hexagon nut self-locking M5	6	Pc.
10	3201-7910	Wood wedge size 2 with PU-foam	1	Pc.
	3203-7810	Wood wedge size 2 with PU-foam	1	Pc.
11	1028-0430	Pressboard screw countersunk head 4x30	4	Pc.
12	3213-7831-015	A-wedge mounting adapter black	1	Pc.
13	3231-7832	Arresting part for A-wedge	1	Pc.
14	1021-0516	Hexagon socket head cap screw M5x16	2	Pc.
15	1030-0514	Button head socket screw with flange M5x14	2	Pc.
	3201-7940-323	Cover for abduction block size 2 iron grey	1	Pc.
	3203-7840-323	Cover for abduction block size 3, iron grey	1	Pc.

HOGGI GmbH
Eulerstraße 27 • 56235 Ransbach-Baumbach • Deutschland
Telefon: (+49) 2623 / 92 499-0 • Telefax: (+49) 2623 / 92 499-99
E-mail: info@hoggi.de • www.hoggi.de
